

OPEN

ANALYTICA

AutoStart[®]

STERILE SINGLE-USE 150 mL BURETTE

Setting Up

1. Close the WHITE On/Off clamp (D), and BLUE bypass clamp (C).

2. Open filtered vent (E). NOTE: This vent should be left open during normal use.

3. Remove the spike cap (A)

4. Puncture solution container with spike (B).

5. Open WHITE on/off clamp (D). Fluid will begin to fill the chamber and will be stopped by the float (G) at approx 7-10mL.

6. Open the spike port cap (H).

7. Connect an infusion line to the spike port (H).

8. Prime the system according to the infusion line instructions.

- The device is now ready for use in AutoStart mode.

- To disable AutoStart mode, close WHITE on/off clamp (D)

Medication Delivery

1. Open and shut the BLUE bypass Clamp (C) to fill the Burette to the desired level.


2. Add medication via injection site (F) as per hospital protocol.

- If WHITE on/off clamp (D) is open, the AutoStart float (G) will automatically return the device to continuous infusion mode once the chamber has emptied.

⚠ Important Notes




- Safe for use above pressure infusion equipment.
- The float (G) shuts off flow once the fluid reservoir is empty. This shutoff is not for long-term use.
- Replace device every 24 hours or as per hospital protocol.
- Sterile whilst packaging intact. Do not use if packaging is damaged or if protective caps are not in place.
- Use aseptic technique.
- **WARNING:** Air in infusion line may cause embolism.

 NON-PYROGENIC
NON-TOXIC
LATEX FREE

0123

STERILE EO

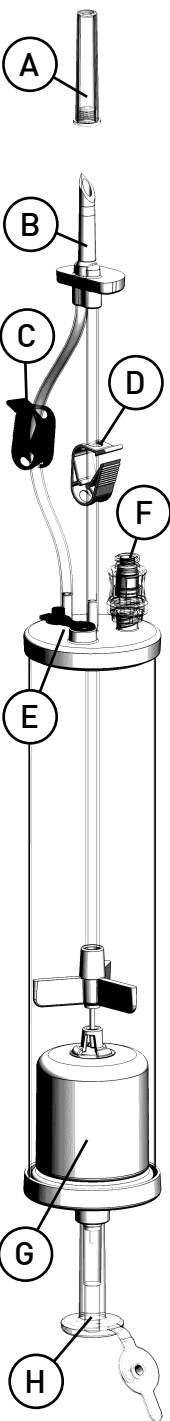

Manufacturer: Zhejiang Lingyang Medical Apparatus Co. Ltd. Linhai, Zhejiang, CHINA


TGA Sponsor: Analytica Ltd. 85 Brandl St. Eight Mile Plains, Brisbane, 4113 AUSTRALIA

www.AutoStartBurette.com

ISO 8536-5 - BS - 150

REF 0080

LOT

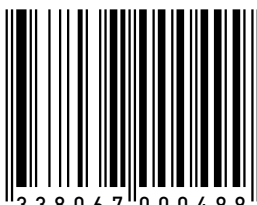

9 338067 000499